



**FRESENIUS
KABI**

caring for life

How to connect the Freka® Belly Button extension set


Instructions for use



Remove the Freka Belly Button extension set from its packaging.




Open the sealing tab of the Freka Belly Button. Line up the black lines on the Freka Belly Button and the Freka extension set.

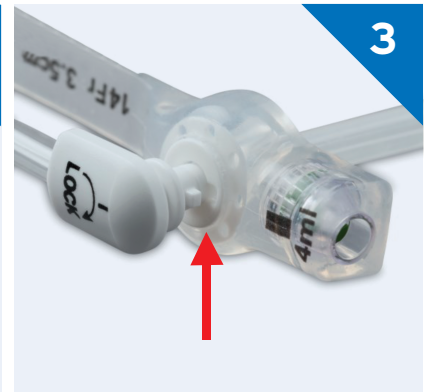


Ensure the extension set is correctly lined up with the feeding port on the Freka Belly Button, before pushing the end of the extension set into the feeding port.



Gently rotate the extension set 90° clockwise, as indicated on the connector.

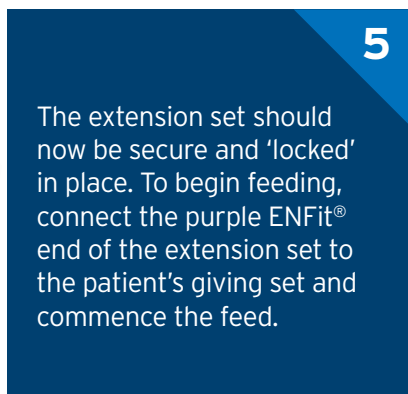

The extension set should now be secure and 'locked' in place. To begin feeding, connect the purple ENFit® end of the extension set to the patient's giving set and commence the feed.

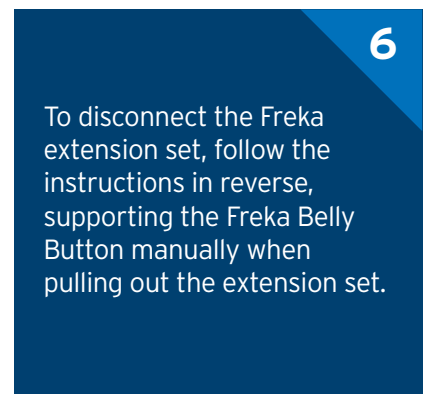

To disconnect the Freka extension set, follow the instructions in reverse, supporting the Freka Belly Button manually when pulling out the extension set.